

ViSiGi LUX™

Frequently Asked Questions

Q: Will the suction work the same as my standard ViSiGi®?

A: Yes. ViSiGi LUX™ provides the same trusted flexibility and suction performance as the ViSiGi®, now with an added benefit of near-infrared illumination for enhanced visibility.

Q: Will this work with my current imaging system?

A: Any imaging system capable of capturing the near-infrared (NIR) spectrum will work with ViSiGi LUX™. A simple way to confirm compatibility- if your system can detect ICG, you will be able to use ViSiGi LUX™.

Q: How do I know if the device is on?

A: The device is shipped in the "on" position. Once the activation tab is removed, a red light on the tip indicates that the lights are on.

Q: Do I need to position the device a certain way during insertion to see the lights?

A: The LEDs provide circumferential illumination due to the unique material used in ViSiGi®, allowing visibility from all directions—so orientation during insertion is not critical.

Q: Can I do an ICG leak test with this device?

A: Yes, the on/off switch allows you to deactivate the lights when performing an ICG leak test.

